

APPENDIX 4: BD Guidance to Changing Date and Time Setting



Date and time setting

BodyGuard™ T syringe pump

Backup Battery Depletion

If the internal backup battery has been depleted, the pump displays an alarm and requires to enter the current date and time.

To enter date and time, do as follows:

1. Press the **⏏** key.

Time & Date
Incorrect date/time
Press ▶ to restore

4. Press the **⏏** key or the **⏏** key to change date, then press the **⏏** key.

16.08.2020 10:30:45
Date 16
Change ▲▼, Press ▶

2. Press the **⏏** key or the **⏏** key to change year, then press the **⏏** key.

16.08.2020 10:30:45
Year 20
Change ▲▼, Press ▶

5. Press the **⏏** key or the **⏏** key to change hour, then press the **⏏** key.

16.08.2020 10:30:45
Hours 10
Change ▲▼, Press ▶

NOTE: Press the **⏏** key to go back to the previous step.

3. Press the **⏏** key or the **⏏** key to change month, then press the **⏏** key.

16.08.2020 10:30:45
Month 08
Change ▲▼, Press ▶

6. Press the **⏏** key or the **⏏** key to change minutes, then press the **⏏** key.

16.08.2020 10:30:45
Minutes 30
Change ▲▼, Press ▶

CE
0344

Becton, Dickinson U.K. Limited, 1030 Eskdale Road,
Winnesh Triangle, Wokingham, RG41 5TS

bd.com/en-uk

BD, the BD Logo and BodyGuard are trademarks of Becton, Dickinson and Company or its affiliates. © 2021 BD. All rights reserved. (BD-2021-1, July 2021)



Permission from BD to use